







GENERAL HANDPIECE MAINTENANCE & CLEANING

*** Maintenance for after use and before autoclaving ***

1 Avroe Crescent
The Pavilions, Blackpool
Lancashire, FY4 2DP
t: 01253 404 774
e: repairs@mcrepairs.co.uk
w: www.mcrepairs.co.uk



STEP 1	STEP 2	STEP 3	STEP 4	STEP 5	STEP 6
					
PREPARATION	CLEAN	CONNECT	SPRAY	CHECK	STERILISE
<ul style="list-style-type: none"> • Detach handpiece from tubing. • Remove bur. • DO NOT submerge handpiece in any solution. 	<ul style="list-style-type: none"> • Wipe away any external debris using a non-alcoholic, approved disinfectant wipe. • DO NOT use any chemical solution or submerge in water. • Using a dry tissue, remove any excess detergent leftover on the handpiece from the wipe. 	<ul style="list-style-type: none"> • Select the correct oil adaptor for the handpiece you want to lubricate. • Ensure nozzle is securely attached. • Shake the spray can well before attaching to the handpiece. 	<ul style="list-style-type: none"> • Insert the oil adaptor into the handpiece. • Always keep the spray can in an upright position. • Spray lubrication through the handpiece for 2-3 seconds into a clean tissue. * Please note, if oil comes through the handpiece head and has a brown discolouration, continue to spray the handpiece 2 more times until it comes through clear. 	<ul style="list-style-type: none"> • Repeat steps 3 & 4 until the oil comes through clear. * If it does not come through clear after 3 sprays, send off for a service. This discolouration is due to a rusting issue. (If the oil comes through the head of the handpiece and has a black discolouration, continue with the remainder of the sterilisation process and send the handpiece for a service before it can be used again). • Proceed to Step 6. 	<ul style="list-style-type: none"> • Package in accordance with applicable standards. • Place your handpiece in the autoclave ensuring you have selected the correct cycle and you have checked that your autoclave does not surpass 138°C.

CAUTION:

- DO NOT** use chemical disinfectants, abrasive cleaners or antibacterial soaps as a cleaner.
- DO NOT** submerge into liquids.
- DO NOT** use ultrasonic cleaners.
- DO NOT** operate or run a handpiece without a bur correctly installed.
- DO NOT** exceed 138°C when autoclaving.

Please note, these are general guidelines as recommended by MC Repairs Ltd. For exact guidelines, you should consult your handpiece user manual.



MCRepairs LTD

The handpiece specialists